

How to Use GammaCore™

SETUP



Clean the treatment site with an alcohol swab



Remove the Caps



Locate the Treatment Site

Find the pulse on the right side of the neck (carotid pulse)



Apply Conductive Gel

Apply a pea sized amount of gel to the stimulation surfaces

TREATMENT



Turn on the GammaCore Device



Place Device on Right Side of Neck

Slowly roll the thumbwheel until the patient feels slight muscle twitching or to maximum level tolerated by patient



Apply Treatment for 90 Seconds



Post-Treatment

Turn the Device Off
Clean the gel off of the device and neck
Wipe the device with an alcohol swab

The GammaCore Device is intended for use as a non-invasive neurostimulator using proprietary electrical signals to treat migraines.

Do NOT use the GammaCore device if you have an implanted device such as a pacemaker, hearing aid implant or any implanted electronic device, or if you have diagnosed narrowing of the arteries or a bilateral or right cervical vagotomy

These instructions are NOT intended to be a replacement for the Instructions for Use. Please refer to the Instructions for Use for full prescribing information